TENS/EMS device

TENS/EMS-apparat

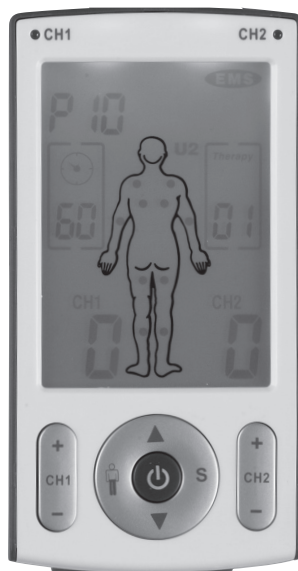
TENS/EMS-laite

Manual

Bruksanvisning

Brukerveiledning

Käyttöohje

Art.no

34-4103

Model

LT3011

Ver. 201201

clas ohlson

English

Svenska

Norsk

Suomi

TENS/EMS device

Art. No 34-4103 Model LT3011

Please read the entire instruction manual before use and save it for future reference. We reserve the right for any errors in text or images and for making any necessary technical changes to this document. If you should have any questions concerning technical problems please contact our Customer Services.

1. Safety

- This product is intended for home use.
- Read, understand and follow the user advice in these instructions. Bear in mind the restrictions and risks that accompany this device. Read the cautionary texts and symbols on this device.
- This product may not be used in the following circumstances:
 - Confirmed or suspected heart defects.
 - Some persons may experience skin irritation or hypersensitivity from electric stimulation or from the use of gels.
 - If a rash appears or pain persists, suspend treatment and consult a doctor.
- Placement of electrodes and settings should be based on advice from a doctor or a person who is used to administering treatment involving similar devices.
- Local skin irritation may occur from prolonged use of electrodes in the same position.
- Do not use electrodes on swollen, infected, inflamed or damaged skin. Make sure the electrodes have good contact with the skin.
- If the stimulation level is uncomfortable and too high for you the impulses must be reduced to a more agreeable level. Suspend treatment and consult a doctor if you are unsure.
- This product should not be used by persons who are regularly out in traffic or persons who monitor machines and processes.
- Do not use this product on premises where aerosols or oxygen are stored or used.
- Do not use this product in the vicinity of explosive, highly flammable substances or gases.
- Do not use this product at the same time with other products that send electric impulses into the body.
- Do not connect electrodes to any other device or connect any other device, i.e., earphones and the like, to the TENS/EMS device.
- Do not use sharp objects to press buttons.
- Make sure that electrodes and cables are undamaged before using.
- TENS/EMS device may only be used in conjunction with authorised accessories.
- Treatment may not be applied above the neck, i.e., to the face, on the throat, or anywhere on the head.

This product must not be used in the following circumstances:

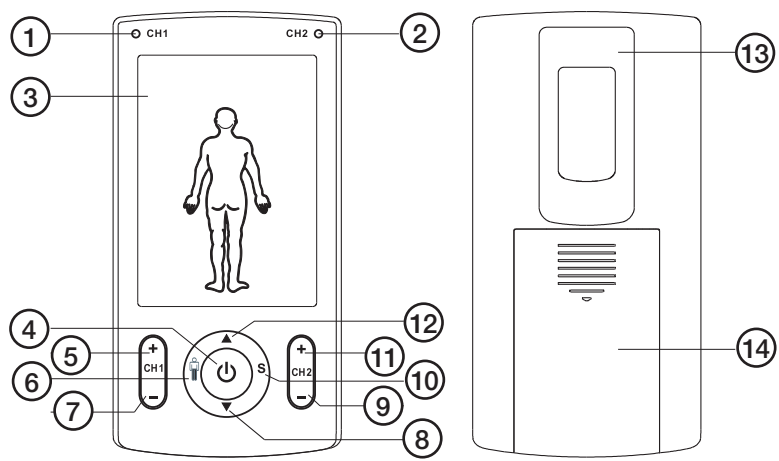
- Pregnant women should consult their doctors before using.
- This product should not be used if carcinogenic changes are found in the area treated.
- Treatment should not be applied to swollen, infected, inflamed or damaged skin.
- This product should not be used on persons who have a pacemaker or defibrillator.
- This product should not be used on persons who have diabetes or abnormally high blood pressure.
- On persons who have epilepsy.
- Serious circulation problems in the body's lower parts.
- Hernia in the groin or abdominal area.
- In cases of known myocardial inflammation.
- Do not use this product for symptoms that have not been diagnosed.

Caution! Do not use this product if you are unsure how it works. Consult a doctor or a person who is used to treatment using products of a similar kind if you are unsure.


2. The package contains

- TENS/EMS-apparatus
- 2x electric cables
- 4x electrodes (40 x 40 mm)
- User manual
- 4 x AAA/LR03


3. Description of Product

1. **CH1** – Light diode that lights up when channel 1 is activated.
2. **CH2** – Light diode for channel 2 that lights up when channel 2 is activated.
3. LCD display
4. **[⏻] Switch**. Press to switch on, press and hold down to switch off.
5. **[CH 1+]** – Increases intensity for channel 1. From **0** (switched off) to **60** (max).
6. **[👤]** – Select user **U1** (user 1) or **U2** (user 2).
7. **[CH 1-]** – A. Switch off button lock. B. Reduces intensity for channel 1. From **60** (max) to **0** (switched off).
8. **[▼]** – Programme selection, back. **EMS** P20 – P01, **TENS** P20 – P01, **U** (own settings) U10 – U01, **MASSAGE** P10 – P01.
9. **[CH 2-]** – A. Closes button lock. B. Lowers intensity for channel 2. From **60** (max) to **0** switched off).
10. **[S]** – A. Press and hold for ca 3 seconds to set the parameters for use in U-position. B. Press once for pause, press again to continue.
11. **[CH 2+]** – Increases intensity for channel 2. From **0** switched off) to **60** (max).
12. **[▲]** – Programme selection, forward. **EMS** P01 – P20, **TENS** P01 – P20, **U** own settings) U01 – U10, **MASSAGE** P01 – P10
13. Belt clasps
14. Battery cover
15. Connection for channel 1
16. Connection for channel 2

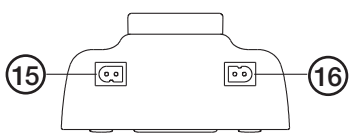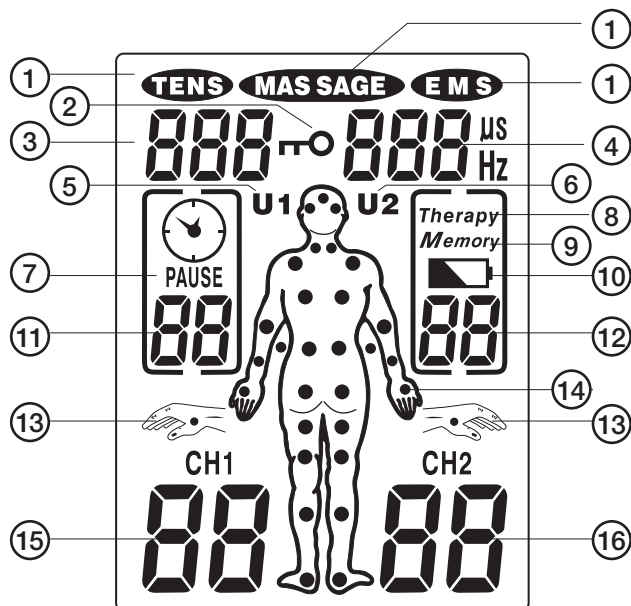4. Display

1. TENS, MASSAGE or EMS show which method of treatment is being used.
2. The key shows that the button lock is activated. The buttons for increasing intensity (5) and (11) are locked. The button lock is activated automatically after a certain time.
3. Programme numbers P01 – P20 (pre-installed treatment programme) U01 – U10 (programme for own installation).
4. Shows installed pulse frequency (Hz) and pulse time (µs).
5. U1 user 1.
6. U2 user 2.
7. Pause
8. Treatment symbol.
9. Memory control.
10. Battery status.
11. Time of treatment.
12. Number of treatments or memory.
13. Site of treatment on hand.
14. Area of treatment on body (does NOT show where electrodes should be placed on the body).
15. Channel 1 intensity.
16. Channel 2 intensity.

5. Buttons and functions

Functions

TENS (Transcutaneous Electrical Nerve Stimulation) – Electrotherapy with low-intensity and short impulses to relieve pain, for example.

EMS (Electrical Muscle Stimulation) – Electric stimulation that boosts blood circulation.

MASSAGE – Electric stimulation that stimulates muscles in the treated area to mild activity which in turn results in an improved sense of well-being.

Button functions


	In treatment position	<ol style="list-style-type: none"> 1. Switch for switching on the device and to start treatment. 2. Press to stop treatment. 3. Press and hold down button 3 seconds to switch off.
	In setting position	Confirm settings and return to normal display.
[▲]	In treatment position	Select treatment programme forward. EMS (P01 – P20) – TENS (P01 – P20) – U (own settings U01-U10) – MASSAGE (P01 – P10).
	In settings position	Press button to change settings for user 1 or user 2.
[▼]	In treatment position	Select treatment programme, back. EMS (P20 – P01) – TENS (P20 – P01) – U (own settings U10-U01) – MASSAGE (P10 – P01).
	In settings position	Press button to change settings for user 1 or user 2.
	In treatment position	Select user 1 or user 2.
	In setting position	Select user 1 or user 2.
[S]	In treatment position	Press button for pause and to resume treatment.
	In setting position	Press button to select settings programme and go to next setting.
[CH1-]	In treatment position	<ol style="list-style-type: none"> 1. Reduce output intensity for channel 1. 2. Press to unlock button lock.
	In setting position	<ol style="list-style-type: none"> 1. Press and hold both [CH 1-] and [S] for 3 seconds to lock settings programme 2. Press and hold [CH 1-] and  at the same time for 3 seconds to restore settings for user 1 or user 2.
[CH2-]	In treatment position	<ol style="list-style-type: none"> 1. Reduce output intensity for channel 2. 2. Press [▼] to unlock button lock.
	In setting position	<ol style="list-style-type: none"> 1. Press and hold both [CH 2-] and [S] for 3 seconds to open memory function (in U-position).

[CH1+]	In treatment position	<ol style="list-style-type: none"> 1. Start treatment. 2. Raise output intensity for channel 1. 3. First press [CH1 -] or [CH 2 -] to open button lock (if it has been activated).
[CH2+]	In treatment position	<ol style="list-style-type: none"> 1. Start treatment. 2. Raise output intensity for channel 2. 3. First press [CH1 -] or [CH 2 -] to open button lock (if it has been activated).

6. Getting started

6.1 Inserting batteries

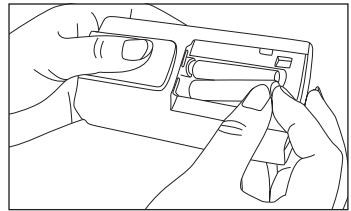
1. Open battery cover (on back) by applying pressure to it and at the same time push cover in the direction of the arrow.
2. Insert 4 AAA/LR03-batteries into battery compartment as indicated by markings.

6.2 Attaching cables to electrodes


Insert cable contacts in electrode contacts. Squeeze the contacts together so that no metal is visible at any connection.

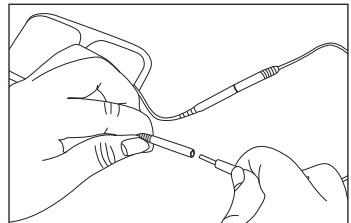Caution! Use only electrodes approved as being in conformity with: IEC/EN60601-1 and IEC/EN60601-1-2.

6.3 Attaching electrode cables to the TENS/EMS device

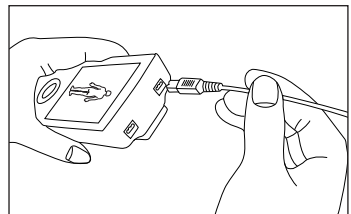
1. First make sure that the device is switched off.
2. Grasp an insulated part of the contact and connect it to one of the outlets (15) and (16) on the topside.
3. You can use a channel with two electrodes, or both channels and so four electrodes at the same time. If you use both channels you can stimulate two different areas of the body at the same time.

6.4 Placing the electrodes on the skin

For hygienic reasons electrodes should be used exclusively by one and the same person. Extra plates are available as accessories (our Art. No. 34-4103-1).

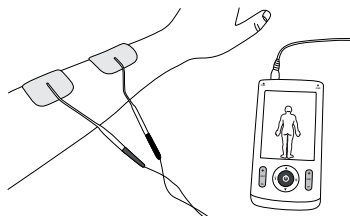Make sure that is skin is completely dry and clean. Press the electrodes tightly against the skin so that there is full contact over the entire surface.

Caution!

- Wash, degrease, and dry skin before attaching self-adhering electrodes.
- Do not start the TENS/EMS device before electrodes are firmly in place on the skin.
- Never remove electrodes from the skin when the TENS/EMS device is switched on.

6.5 Switching on TENS/EMS device

Before you switch on the device we recommend that you read through the cautionary texts, especially under what circumstances the device should not be used. The TENS/EMS device is not a toy but a powerful auxiliary.

Switching on the device: Press and hold down  until the display lights up.

6.6 Selecting user

There is memory for two different users (U1) and (U2) so that they can use this product with their own individual settings.

Press  to select user 1 or 2.

6.7 Selecting type of treatment

There are two different working positions: **Normal treatment** (med preset program) and **U-position** (with possibility of using ones own settings).

A. Normal treatment

1. Press  to open normal position (P). The display shows:

“TENS” – shows that TENS-treatment has been selected.

“P01” – shows that programme 1 (of 20) is installed.

“Clock” – blinks while treatment is being applied.

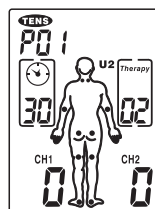
“30” – shows installed treatment time.

“U2” – that the setting is valid for user 2.

“Therapy” – shows how many treatments have been administered.

“CH1 0” – shows that channel 1 has not been activated (0 intensity). When the channel is activated the indicator lamp CH1 in top left corner lights up

“CH2 0” – shows that channel 2 has not been activated (0 is intensity). When the channel is activated the indicator lamp CH2 in upper right corner lights up.

The dots on the body diagram show the area of treatment and NOT where the electrodes are to be placed on the body.

Normal position

2. Use pre-installed programme:

Press [▲] or [▼] several times to select desired type of treatment. The different types of treatment all have pre-installed programmes.

EMS (Electrical Muscle Stimulation) with 20 different pre-installed programmes for treatment on different places on the body. The different programmes are shown on the display.

TENS (Transcutaneous Electrical Nerve Stimulation) with 20 different pre-installed programmes for treatment on different places on the body. The different programmes are shown on the display.

MASSAGE (electric stimulation of muscles in the treatment area) with 10 different pre-installed programmes for treatment on different places on the body. The different programmes are shown on the display.

3. Activate the programme selected by pressing [CH1+] or [CH2+]. Press several times to increase intensity (0–60).

B. Treatment with own programme settings

U-position (own programme settings) Possibility in 10 different programmes (U01–U10) of applying one's own settings for intensity, impulse frequency, pulse duration and treatment time.

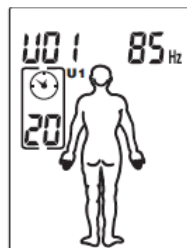
Install your own settings as follows:

1. Press [▲] or [▼] several times to install the desired programme (U01–U10) at U-position.
2. Select user 1 or user 2 with .
3. Press and hold [S] for 3 seconds to open setting position.
4. The treatment time blinks. Set the desired time (5–90 minutes) with the arrows.
5. Press [S] to confirm.
6. Set pulse frequency (2–120 Hz) and pulse duration (50–300 µs) in the same way.
7. Press [S] to confirm. The installed pulse frequency and pulse duration appear alternately on the display.

A user setting can be easily locked in by a doctor or therapist. A locked setting is shown as L01 – L10 (L = locked) on the display.


To lock or unlock one's own setting, proceed as follows:

1. Remove batteries.
2. Hold down  and quickly insert batteries.
3. Release  when batteries are in place.
4. The display shows:

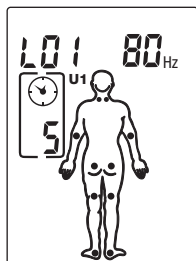- Press and hold [S] and [CH 1-] at the same time for ca 3 seconds to lock or open settings in "U-position". A locked setting appears as L 01 – L10 (L = locked) on the display.

The display shows:

To save and return, press and hold  until the device has switched off.

6.8 Selecting treatment programme

Press [] or [] repeatedly to install the desired programme (TENS – EMS – MASSAGE – USER). Consult a doctor if you are unsure which programme is best for you.

6.9 Starting treatment

- Press [CH1 +] or [CH2 +] to increase output intensity for channel 1 or channel 2.
- Press [CH1 -] or [CH2 -] several times to reduce output intensity for channel 1 or channel 2. Adjustment occurs in 1 mA steps.

6.10 Adjusting treatment intensity

Hold down button [CH1 +], [CH2 +], [CH1 -] or [CH2 -] to adjust output intensity. Hold button down until you reached recommended intensity. Proceed in the same way for the other channel if it is to be used.

Caution! Reduce stimulation level if you experience discomfort. Contact the doctor if the discomfort persists.

6.11 Making a pause in treatment

- Press  to break off treatment.
- Press [S] to make a pause in treatment. Press again to continue.

6.12 Button lock

If no buttons are used for 10 seconds the button lock is activated automatically. Press [CH1 -] or [CH2 -] to open the button lock.

6.13 Switching off

1. Hold down  for ca 3 seconds to switch off.
2. If nothing happens for 2 minutes (in treatment position) the device shuts off automatically.
3. Remove electrodes. If treatment is to be continued within a short time, you may leave the electrodes on the body.
4. Wipe the skin clean with mild soap and water after the electrodes have been removed. Contact your doctor if the skin has become irritated.

6.14 Changing batteries

Open battery cover and remove batteries. Insert 4 new AAA/LR03 batteries following the markings in the battery compartment.

6.15 Battery warning

When the battery symbol starts blinking it is time to replace the batteries. The device will continue working for a few hours before it shuts off.

6.16 Checking the saved data in memory

In the locking/unlocking position for one's own settings (U-position) it is possible to display data stored in memory.

- Press and hold both [CH 2-] and [S] for 3 seconds to check the stored data.
- Press [] or [] to monitor new incoming or preceding data.
- Press and hold [S] for 3 seconds to delete all data from memory.
- Press and hold  until the display turns of.

7. EMS muscle training programme

Sports training

Electric muscle stimulation is being used by more and more sports activists to improve their capabilities, achieved by increasing muscle strength, volume and endurance.

- **TENS** (Transcutaneous Electrical Nerve Stimulation) – Electrotherapy with low-intensity, short impulses to relieve pain, for instance.
- **EMS** (Electrical Muscle Stimulation) – Electric muscle stimulation that boosts blood circulation.
- **MASSAGE** – Electric stimulation to induce mild activity in the area of treatment, which leads to an improved sense of well-being.

8. Recycling

In disposing of this product please comply with local regulations. If you are unsure how to proceed, contact your local authorities.

9. Specifications

Material	ABS
Channels	Two channels
Power supply	4x AAA/R03-batteries
Wave form	Two-phase quadrilateral wave
Pulse length	50–300 μ s
Pulse frequency	1–150 Hz
Treatment time	5–90 minutes
Intensity	Adjustable from 0 to 60 mA (at 1000 Ohm)
Environment of use	5 °C to 40 °C at 30–75 % RH and air pressure 700–1060 hpa
Storage	-10 °C to 55 °C at 20–93 % RH and air pressure 700–1060 hpa
Size	117 x 60 x 25 mm (excluding belt clasps)
Weight	110 g (without batteries)

Caution! Read all instructions before using.

Unit type BF.

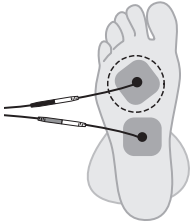
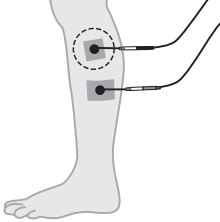
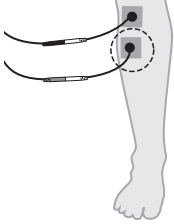
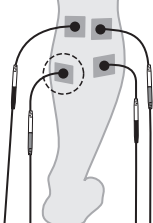
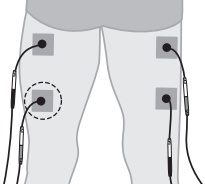
This product should be handled in accordance with decree 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment).






Complies with the European Medical Device Directive (93/42/EEC) and amended by directive 2007/47/EC requirements. Notified body TÜV Rheinland (CE0197)

This product meets the requirements set down in the following European standards: EN 60601-1, EN 60601-1-2, EN 60601-1-4, EN 60601-2-10, ISO10993-5, ISO10993-10


10. Placement of electrode plates

Placement of electrodes	Treatment position
	<p>In upright sitting position with feet resting on floor.</p>
	<p>Upright sitting position with feet resting on floor.</p>
	<p>Upright sitting position with feet under a piece of furniture to avoid bending of ankles.</p>
	<p>Upright sitting position with support for feet and back (e.g., sitting in a door frame).</p>
	<p>Lying flat on one's stomach with ankles comfortably immobilised.</p>

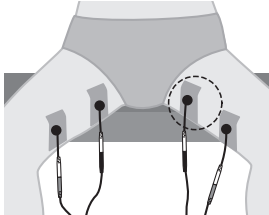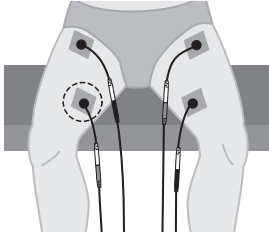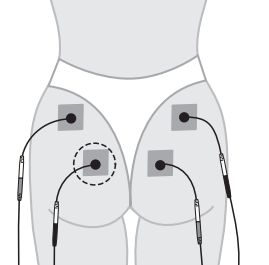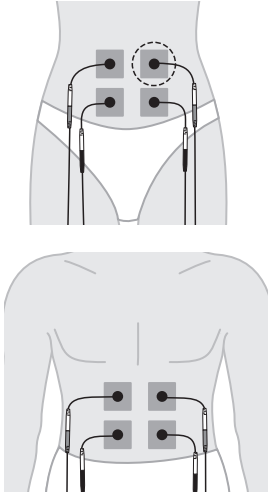
	<p>Upright sitting position, knees pressed together around a hard (but comfortable) object.</p>
	<p>Upright sitting position, knees pressed together around a hard (but comfortable) object.</p>
	<p>Lying on one s stomach or standing up.</p>
	<p>Lying on one s back stretched out.</p>

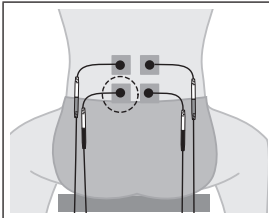
Upright sitting position.

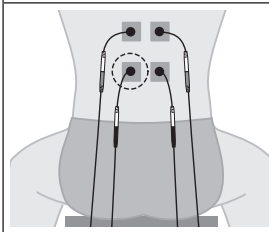
Upright sitting position.

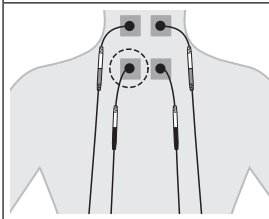
Upright sitting position.




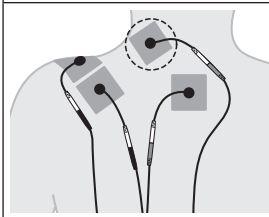
Upright sitting position.

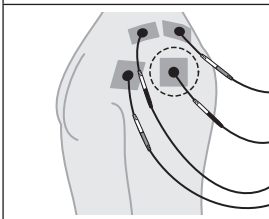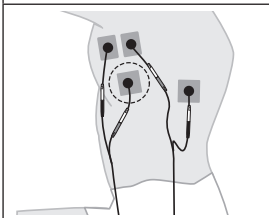Upright sitting position with elbows pressed against side of the body (inside chair armrests).

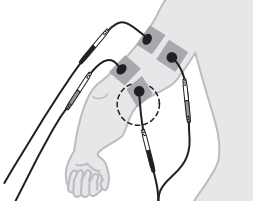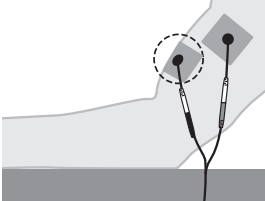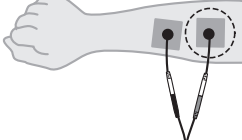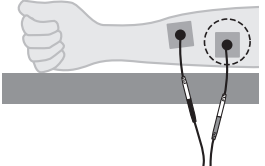Upright sitting position with elbows pressed away from the body (outside chair armrests).

	<p>Upright sitting position with arms resting on chair armrests.</p>
	<p>Upright sitting position with lower arms resting on chair armrest and palms of hands upward.</p>
	<p>Upright sitting position with lower arms and palms resting on chair armrests.</p>
	<p>Upright sitting position with lower arms resting on chair armrests) and palms of hands upward. Hands clasping some object so fingers are slightly bent.</p>

11. Treatment diagram

Program		1	2	3	4	5
Indication		1. Tension headache 2. Migraine 3. Back pain 4. Pain in hip 5. Pain in knee joint	1. Sciatica 2. Back pain 3. Pain in hip	1. Muscle cramps	1. Muscle cramps 2. Improve capillary function	1. Tension headache 2. Migraine 3. Back pain 4. Pain in hip 5. Pain in knee joint
Treatment position	Hand	•				•
	Nape of neck	•		•		•
	Shoulder					
	Back		•		•	
	Chest					
	Upper arm			•		
	Elbow	•				•
	Arm			•		
	Hip	•	•		•	•
	Abdomen					
	Rear	•	•		•	•
	Hip joint	•	•		•	•
	Thigh		•	•	•	
	Knee	•				•
	Calf			•		
Foot						

Program	6	7	8	9	10	
Indication	1. Tension headache 2. Migraine 3. Pain in back 4. Pain in hip 5. Pain in knee joint 6. Muscle cramps	1. Bruise, strain, wrenching and sprain. 2. Inflammation in upper arm 3. Pain in upper arm/shoulder blade	1. Tension headache 2. Pain in back of neck 3. Pain in shoulder 4. Pain in arm 5. Pain in elbow 6. Pain in hip 7. Pain in knee joint	1. Bruise, strain, wrenching and sprain. 2. Inflammation in upper arm 3. "Frozen shoulder"	1. Muscle pains 2. Pain in elbow 3. Pain in hip 4. Pain in knee joint	
Treatment position	Hand	•		•		
	Nape of neck	•	•	•	•	
	Shoulder		•	•	•	
	Back					
	Chest					
	Upper arm		•	•	•	
	Elbow	•		•		•
	Arm			•		
	Hip	•				
	Abdomen					
	Rear	•				
	Hip joint	•		•		•
	Thigh		•		•	
	Knee	•		•		•
	Calf		•		•	
Foot					•	

Program		11	12	13	14	15
Indication		1. Pain in nape of neck 2. Muscle pain 3. Pain in elbow 4. Pain in hip 5. Pain in knee joint	1. Sports injuries: bruises, strain, sprain, and wrenching 2. Pain in elbow 3. Pain in nape of neck 4. Muscle pain	1. Tension headache 2. Migraine 3. Pain in lumbar region 4. Pain in hip 5. Pain in knee joint	1. Tension headache 2. Pain in nape of neck 3. Pain in shoulder 4. Pain in arm 5. Pain in elbow 6. Pain in hip 7. Pain in knee joint	1. Tension headache 2. Pain in nape of neck 3. Pain in shoulder 4. Pain in arm 5. Pain in elbow 6. Pain in hip 7. Pain in knee joint
Treatment position	Hand			•	•	•
	Nape of neck	•	•	•	•	•
	Shoulder		•		•	•
	Back		•			
	Chest					
	Upper arm		•		•	•
	Elbow	•	•	•	•	•
	Arm				•	•
	Hip		•	•		
	Abdomen		•			
	Rear		•	•		
	Hip joint	•		•	•	•
	Thigh		•			
	Knee	•		•	•	•
	Calf		•			
Foot	•	•				

Program	16	17	18	19	20	
Indication	1. Tension headache 2. Pain in nape of neck 3. Pain in shoulder 4. Pain in arm 5. Pain in elbow 6. Pain in hip 7. Pain in knee joint	1. Tension headache 2. Migraine 3. Pain in lumbar region 4. Pain in hip 5. Pain in knee joint	1. Sports injuries: bruises, strain, sprain, wrenching 2. Inflammation in upper arm 3. "Frozen shoulder"	1. Tension headache 2. Migraine 3. Pain in lumbar region 4. Pain in hip 5. Pain in knee joint	1. Sciatica 2. Back pain 3. Pain in hip	
Treatment position	Hand		•		•	
	Nape of neck	•	•	•	•	
	Shoulder	•		•		
	Back				•	
	Chest					
	Upper arm	•		•		
	Elbow	•	•		•	
	Arm	•				
	Hip		•		•	•
	Abdomen					
	Rear		•		•	•
	Hip joint	•	•		•	•
	Thigh			•		•
	Knee	•	•		•	
	Calf			•		
Foot						

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Shanghai International Holding Corp. GmbH (Europe)
Post Clas Ohlson AB, 793 85 INSJÖN
Osoite Clas Ohlson Oy, Annankatu 34–36 A, 00100 HELSINKI